

FUJIFILM

Code No. 995-06101

Autokit Micro Albumin Calibrator Set

Intended use

The Autokit Micro Albumin Calibrator Set, which contains 5 calibrators with different albumin concentration, is designed to be used with Autokit Micro Albumin test for the determination of urinary albumin.

Reagents and ingredients

Micro Albumin Calibrator 1 (5 μ g/mL)	1 bottle \times 2 mL
Micro Albumin Calibrator 2 (25 μ g/mL)	1 bottle \times 2 mL
Micro Albumin Calibrator 3 (100 μ g/mL)	1 bottle \times 2 mL
Micro Albumin Calibrator 4 (300 μ g/mL)	1 bottle \times 2 mL
Micro Albumin Calibrator 5 (500 μ g/mL)	1 bottle \times 2 mL

Contains human serum albumin and 0.09% sodium azide.

Determination of assigned values

Assigned values : See bottle labels

Determination : albumin concentration was determined by turbidimetric immunoassay using the CRM470 as the primary standard.

Storage condition

Store at 2-10°C. Do not freeze.

Warnings and precautions

- For in vitro diagnostic use.
- Not to be used internally in humans or animals.
- Do not use these calibrators for any other purpose than described herein.
- These calibrators are prepared from human sera which were found to be negative for HBsAg, anti-HIV 1 & 2 antibodies and anti-HCV antibody. Because no test can offer complete assurance that products derived from human sera will not transmit infectious agents, it is recommended that the calibrator should be handled with the same precautions used for patient specimens.
- These calibrators contain sodium azide as a preservative. Sodium azide may react with copper or lead plumbing to form explosive compounds. Even though this product contains minute quantities of sodium azide, drain should be flushed well with a large amount of water when discarding the solution.
- Precipitate in a calibrator may indicate deterioration and when present, the calibrator should not be used.
- UDI (Unique Device Identification) is on the box. Keep box until reagent is finished.

Ordering information

Code No.	Product	Package
995-06101	Autokit Micro Albumin Calibrator Set	5 conc. \times 2 mL

Revision date ; Apr. 1, 2018

Manufactured by

FUJIFILM Wako Pure Chemical Corporation

1-2, Doshomachi 3-Chome, Chuo-Ku Osaka 540-8605, Japan
Tel : +81-6-6203-3749
Fax : +81-6-6203-1917
www.wako-chem.co.jp



Distributed by

FUJIFILM Wako Diagnostics U.S.A. Corporation

1025 Terra Bella Ave., Mountain View, CA 94043 U.S.A.
Tel : 877-714-1924
Fax : 804-271-0449
www.wakodiagnostics.com

18.03.19K04