

FUJIFILM

Code No. 991-06201

Autokit Micro Albumin Control Set

Intended use

The Autokit Micro Albumin Control Set, which contains 2 controls with different albumin concentration, is designed to be used with Autokit Micro Albumin test for the determination of urinary albumin.

Reagents and ingredients

Micro Albumin Control 1 (Target Range = 20-40 μ g/mL) 2 bottles \times 2 mL
Micro Albumin Control 2 (Target Range = 75-105 μ g/mL) 2 bottles \times 2 mL
Contains purified human serum albumin and 0.09% sodium azide.

Determination of assigned values

Assigned values : See bottle labels

Value assignment : The Autokit Micro Albumin control values were assigned by calibrating the assay with Wako's Autokit Micro Albumin Calibrator Set (CRM470 traceable) and running multiple runs of n=21 replicates of the control material. Verification of these values were done through analysis of varying the lots of reagents and calibrators to verify the recovered target mean and range.

Storage condition

Store at 2-10°C. Do not freeze.

Warnings and precautions

- For in vitro diagnostic use.
- Not to be used internally in humans or animals.
- Do not use these controls for any other purpose than described herein.
- These controls are prepared from human sera which were found to be negative for HBsAg, anti-HIV 1 & 2 antibodies and anti-HCV antibody. Because no test can offer complete assurance that products derived from human sera will not transmit infectious agents, it is recommended that the control should be handled with the same precautions used for patient specimens.
- These controls contain sodium azide as a preservative. Sodium azide may react with copper or lead plumbing to form explosive compounds. Even though this product contains minute quantities of sodium azide, drain should be flushed well with a large amount of water when discarding the solution.
- Precipitate in a control may indicate deterioration and when present, the control should not be used.

Ordering information

Code No.	Product	Package
991-06201	Autokit Micro Albumin Control Set	2 \times 2 conc. \times 2 mL

Revision date ; Apr. 1, 2018

Manufactured by

FUJIFILM Wako Pure Chemical Corporation

1-2, Doshomachi 3-Chome, Chuo-Ku Osaka 540-8605, Japan
Tel : +81-6-6203-3749
Fax : +81-6-6203-1917
www.wako-chem.co.jp



Distributed by

FUJIFILM Wako Diagnostics U.S.A. Corporation

1025 Terra Bella Ave., Mountain View, CA 94043 U.S.A.
Tel : 877-714-1924
Fax : 804-271-0449
www.wakodiagnostics.com

18.03.19K03